Technical Bulletin

Technisches Bulletin

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Issue:

Software bug, limiting the intermittend acoustic notification of alarms

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Dear customer,

GS strives towards a constant top quality of their medical products done by permanent supervision of manufacturing. Within this supervision, an error limiting the intermittend acoustic notification of alarms of the above mentioned products has been explored.

If the parameter »ALARM« is set to »SHORT« (see Operators Manual, chapter 6, item »Alarm mode (setting), setting «), an alarm is acoustically indicated by a short tone repeating every 90 seconds until the user quits the alarm. Devices, operated with the above mentioned software discontinue after two times of acoustic notification.

GS already provides a new software fixing this bug. Undertaking the next yearly servicing, your technical service partner will install this software to the devices affected.

If you don't have a maintenance agreement with your **corpuls®** representative in charge and servicing is done by outside companies, please do not neglect to contact your national **corpuls®** representative to make an appointment for software update.

In order to not ignore acoustic notifications it may be advisable to temporarily set the parameter »ALARM« to »ON« (see Operators Manual, chapter 6, item »Alarm mode (setting), setting ). The acoustic notification is now given by a continuous tone first ending with the user's acknowledgment.

We thank you for your understanding to implement this update for continued top quality of our products. In case of questions do not hesitate to contact your national **corpuls®** representative.

Document name and location:	U:\Vorlagen\Technische Vorlagen\Technical_bulletin.dot	Release identification:	1.00	 
Creation date:	2007-08-03	Release date:	2007-03-28	
Originator name:	Carsten Fuchs	Release name:	Klaus Stemple	

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Carsten Fuchs
Vice President, Customer Support
Serviceleiter

Document name and location:	U:\Vorlagen\Technische Vorlagen\Technical_bulletin.dot	Release identification:	1.00	 
Creation date:	2007-08-03	Release date:	2007-03-28	
Originator name:	Carsten Fuchs	Release name:	Klaus Stemple	

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GS Elektromedizinische Geräte G. Stemple GmbH